
ELYDA

THERMAL WELLNESS DEVICE USER MANUAL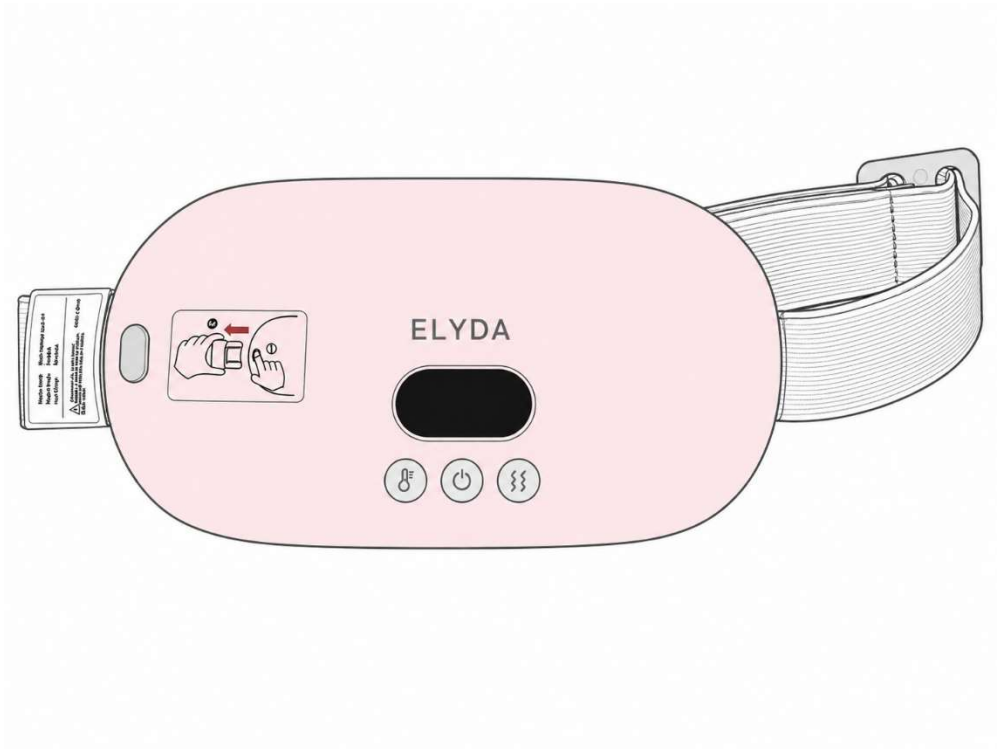

Multilanguage version

Desing by VertexBridgeGlobal. LLC in Wyoming- United States. Manufacturer in China. All rights reserve.

ELYDA – User Manual (English Version)

1. Product Overview

ELYDA Portable Heating & Vibration Belt is designed to provide effective relief from menstrual cramps and abdominal discomfort through advanced heat therapy and vibration massage.

- Ergonomic and lightweight design
 - Adjustable elastic strap
 - LED display with temperature indicator
 - Rechargeable battery
-

2. Product Diagram

Main components:

- Power Button (ON/OFF)
 - Temperature Control Button (TEMP)
 - Vibration Control Button (VIBRATION)
 - LED Display Screen
 - Charging Port
 - Adjustable Belt Strap
-

3. Key Features

Heating Technology

- Graphene-based heating system
- 3 temperature levels:
 - **Low:** 50°C (daily comfort)
 - **Medium:** 55°C (moderate relief)
 - **High:** 60°C (intense relief)

Vibration Modes

- 4 adjustable vibration levels
- Can be used:
 - With heat
 - Without heat

Battery Performance

- Long-lasting rechargeable battery
 - Designed for multiple full sessions (up to 4 cycles of 30 minutes)
-

4. How to Use

Power On / Off

- Press and hold the **ON/OFF button** to turn the device on or off.

Adjust Temperature

- Press the **TEMP button** to switch between temperature levels.

Activate Vibration

- Press the **VIBRATION button** to select intensity level.

Charging

- Use the included USB cable
 - Input: 5V
 - Fully charge before first use
-

5. Safety Instructions

- Do not use while sleeping
 - Do not use for prolonged periods in the same position
 - Stop use if discomfort or skin irritation occurs
 - Not all people have the same sensitivity to heat on their skin. If it feels too hot, use the device over a cotton shirt.
 - Keep away from water and corrosive substances
 - Do not expose to high heat sources
-

6. Contraindications

Do NOT use if:

- You are pregnant
 - You have severe heart conditions
 - You have skin injuries or sensitivity issues in the area
 - You suffer from serious medical conditions without consulting a doctor
-

7. Maintenance & Care

- Clean with a soft, dry cloth
 - Do not immerse in water
 - Store in a cool, dry place
 - Avoid folding the heating area excessively
-

8. Technical Specifications

- Product Name: ELYDA Heating Belt
 - Size: 165 × 85 × 25 mm
 - Material: ABS + Soft Fabric
 - Temperature Levels: 50°C / 55°C / 60°C
 - Charging Voltage: 5V
 - Application Area: Abdomen / Waist
-

9. After-Sales Service

If you experience any issues:

- Contact customer support with purchase details
 - Warranty: up to 6 months
 - Provide:
 - Name
 - Order information
 - Issue description
 - Contact info (Email)
-

10. Tips for Best Results

- Use during the first signs of discomfort

- Combine heat + vibration for stronger relief
- Use in a relaxed position

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